

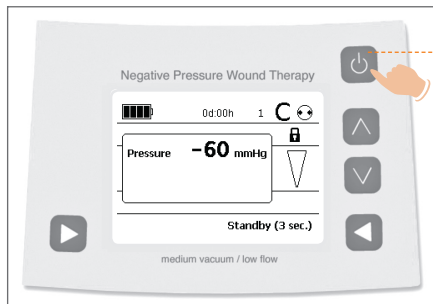
Patient Quickcard Invia® Liberty™

Switch on



Press quickly

Switch off

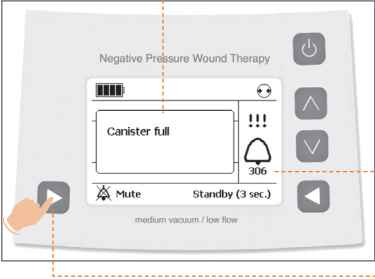


Press and hold for 3 seconds

Press only once

Pump needs approx. 10 sec to switch off

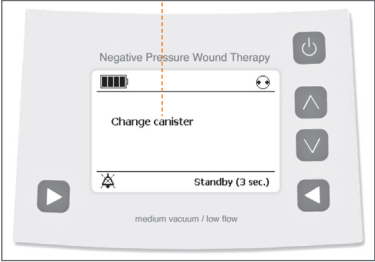
Alarm sounds

- 

1

Problem description appears on display

Fault number

Press to mute
- 

2

Troubleshooting info appears on display
- 3 Please consult with your healthcare provider regarding resolution to any issues or concerns

Medical Vacuum Technology
for Healthcare Professionals

Please contact us or your local Medela representative for details.

Federal law restricts this device to sale or rental by or on the order of a physician.

Medela and Invia are registered trademarks of Medela Holding AG. Liberty is a trademark of Medela, Inc.



Medela AG
Lättichstrasse 4b
6341 Baar / Switzerland
www.medela.com

Medela, Inc.
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
suction@medela.com
www.medelasuction.com