

Invia[®] Foam Dressing Quick Card



This Quick Card does not replace the instructions for use REF 200.8883

Step 1



- Debride any necrotic/eschar tissue, if necessary. Cleanse wound bed and pat dry per protocol.
- Ensure periwound skin is clean and dry. Application of protective skin barrier may protect the periwound skin.

Step 2



- Cut foam the size and shape of the wound and place in the wound cavity.
- Foam should fill the wound cavity and it may be necessary to stack the foam pieces in deep wounds.
- Do not cut foam over wound.

Step 3a



- Cover foam with transparent film.
- Partially peel back one side of layer 1 and then place the adhesive side down over the wound.
- Next, remove the remaining side of layer 1. Do not stretch or pull.

Step 3b



- Now, remove the transparent backing film labeled 2 and then remove the perforated blue tab on the side.
- The transparent film should be adequately anchored to periwound area to maintain an air tight seal.

Step 3c



- The film should extend 3-5 cm beyond the wound margin to facilitate an adequate seal.

Step 4




- Select an appropriate location over the dressed wound bed for the External Suction Interface (ESI) to be applied and pinch the transparent film over the area lifting slightly.
- Cut a small hole in the pinched area (no more than 1cm in diameter).

Step 5



- Peel the backing off of the ESI and center the ESI over the previously cut hole in the dressing. Apply by pressing firmly for adherence.
- Should additional tape be desired it may be applied in a window pane fashion over the perimeter of the transparent film and/or the perimeter of the ESI.

Step 6

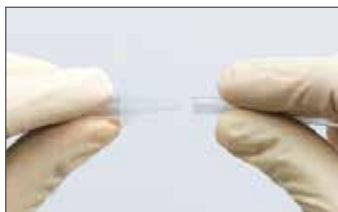

- Connect the ESI Tubing to the pump tubing "conical" connector ensuring that all clamps are open and turn on the pump with the desired physician ordered pressure setting.

Step 7



- Once the pump is on, check that the seal and dressing are secure.
- The finished dressing should be firm to touch.
- If there is a concern of the tubing or ESI creating pressure on the wound margins, utilize bridging technique.

Please contact us or your local Medela representative for details.

CAUTION: Federal US law restricts this device to sale by or on the order of a physician.



Medela AG
Lättichstrasse 4b
6341 Baar, Switzerland
www.medela.com

Medela Inc. USA
1101 Corporate Drive
McHenry, IL. 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
suction@medela.com
www.medelasuction.com

Medela Canada, Inc.
4090B, Sladerview Crescent
Unit 2, Mississauga, Ontario
Canada L5L 5Y5
Phone: 1-905-608-7272
Fax: 1-905-608-8720
info@medela.ca